Hemorrhoid Ligation (Disposable)

Sale training

Technical Services Leader, Medical & Well-being Business

Agenda

- What is Hemorrhoid?
- SCG Hemorrhoid Ligator
- IFU: Intended used /Indication of use/Who eligible to use
- Standard & Regulations
- Clinical Applications
- Package/Price

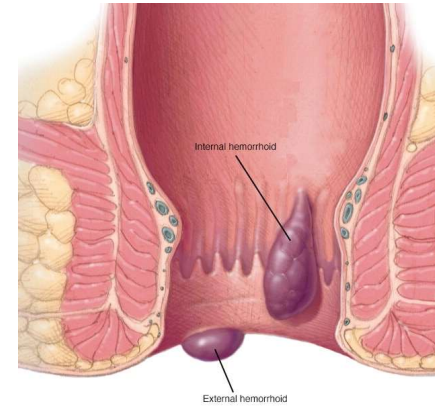
What is Hemorrhoid ?

- Types of Hemorrhoids

Internal Hemorrhoids: Found *inside* the rectum. Typically *not visible* or felt.

External Hemorrhoids: Found under the skin *around the anus*.


Visible and palpable as lumps or bumps.

- Treatment Options

Feature	Internal Hemorrhoids	External Hemorrhoids
Location	Inside the rectum	Under the skin around the anus
Symptoms	Painless bleeding, possible prolapse	Pain, swelling, bleeding, itching
Treatment Options	Creams, stool softeners, ligation, surgery	Topical treatments, sitz baths, laser, surgery
Common Devices	Rubber band ligators, sclerotherapy kits	Laser therapy devices, thrombectomy tools

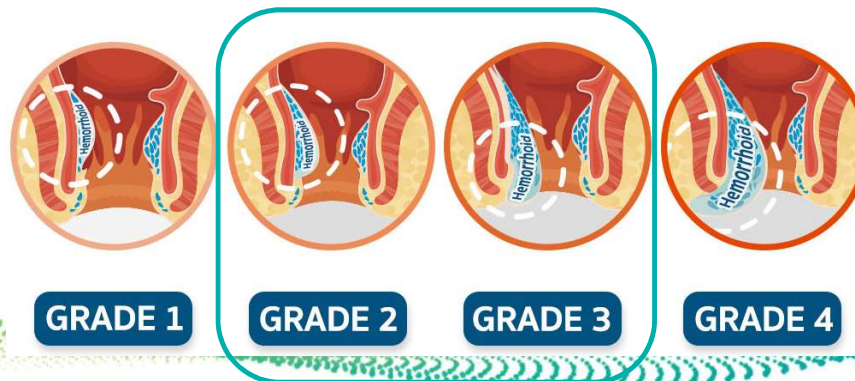- Hemorrhoid Grading System

Competitive landscape

Technic Rubber band only

CRH O'Regan System

McGivney Ligator

Saeed "Short Shot":



Ultroid

Hemorrhoid Ligator

Assistance required	No	Often	Often	Sometimes	No
Length of procedure	Approximately 1 minute	Approximately 5 - 15 minutes	Approximately 5 - 10 minutes	Approximately 10 - 20 minutes	1-3 min
Patient comfort	No Anoscope required, greater patient comfort	Anoscope and technique increase patient discomfort	Technique and multiple site banding increases patient discomfort	Length of procedure and Probe used for procedure increase patient discomfort.	May need anoscope
Ease of Use by Physician	Easy to learn one handed technique	2 handed techniques required, instrumentation is more difficult to utilize	2 handed technique required	2 handed technique required, length of procedure make instrumentation is more difficult to utilize	Easy to learn
Cleaning	Disposable	Sterilization and maintenance required.	Disposable, significantly more waste with suction tubing canisters	Disposable probe, hand piece not disposable and cannot be sterilized	Disposable
Additional Equipment/Capital Investment required	None	Up front instrument purchase Cleaning and sterilizing expense	Wall suction	Power generator	No additional equipment
Training	On-site physician training	None	None	Non-physician onsite and video	On-site/Online
Other Support	Marketing and operation	None	None	Marketing and operational	N/A
Cost	\$65 -75	Initial cost and cleaning and maintenance	\$60 -65 (includes tubing and canisters)	\$80	About \$10-15

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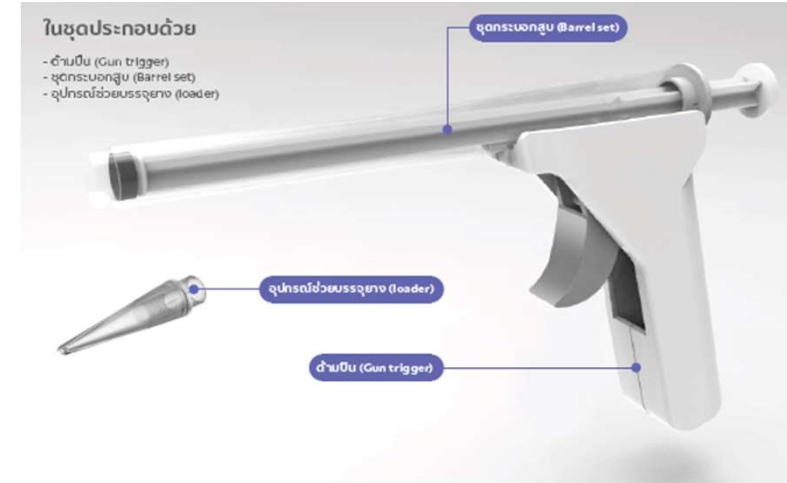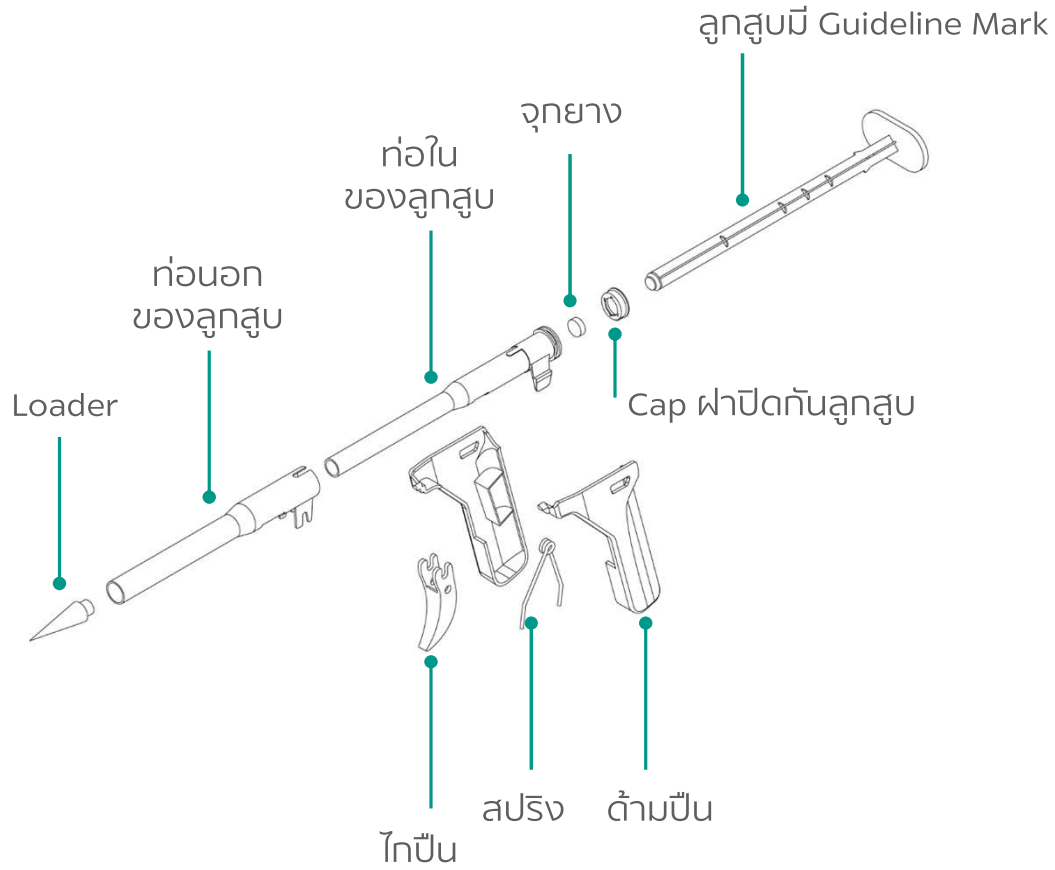Standard & Regulations

- Pre-clinical study-Biocompatibility ISO 10993

Cytotoxicity

FINAL REPORT
Test for *in vitro* cytotoxicity of Hemorrhoid Ligator by elution method as per ISO 10993-5:2009

Study No:
1137/001

RESULTS

The assay was considered valid as the confluency of the cells before treatment was approximately 80%.

Qualitative evaluation

Under microscopic evaluation, the cultures treated with the negative control and vehicle control did not show any cytotoxic response (grade 0), whereas nearly 50% growth inhibition (grade 3) was observed in cultures treated with 0.07 mg/mL concentration of positive control. Therefore, the assay was considered valid.

The cultures treated with the test item extract at different concentrations (30% to 100%) appeared normal without any change in their morphology (grade 0), when compared with the vehicle control.

Quantitative evaluation

The mean absorbance of cells in vehicle control was 0.410, the left and the right mean of the vehicle controls did not differ by more than 15% from the mean of all vehicle controls. The negative control (viability: 96.10%) and positive control (viability: 51.46%) performed as expected.

Cells treated with test item extract at different concentrations 30% to 100% exhibited viability greater than 70% (ranging between 86.59% to 95.85%).

CONCLUSION

Based upon the results obtained in this study performed in line with ISO 10993-5:2009, the given test item, Hemorrhoid Ligator supplied by Texplore Co., Ltd. is considered as non-cytotoxic to BALB/c 3T3 cells.

Sensitization

FINAL REPORT
Skin sensitization test of Hemorrhoid Ligator in guinea pigs (Guinea Pig Maximization Test) as per ISO 10993-10:2021

Study No:
1137/002

RESULTS

Mortality and morbidity

No mortality or morbidity occurred in any of the animals used in this study.

Body weight

A gradual increase in the body weight was observed in all the animals at the end of the experiment. Body weight of animals recorded prior to dosing and end of the experiment are given in Table 1.

Clinical observations

No signs of ill health occurred in any of the animals used in this study.

Grading of skin reactions

Grading of skin reactions performed at 24 h and 48 h after removing the challenge patch are given in Table 2. No sensitization reactions were observed in animals treated with the vehicle controls. No evidence of sensitization was seen in any of the test item treated animals.

CONCLUSION

Based upon the results obtained in this study performed in line with ISO 10993-10:2021, it is concluded that the given test item, Hemorrhoid Ligator, supplied by Texplore Co., Ltd., is considered as non-sensitizer to guinea pigs under the conditions of the present study.

Intracutaneous reactivity

FINAL REPORT
Intracutaneous reactivity test of Hemorrhoid Ligator in New Zealand White rabbits as per ISO 10993-23:2021

Study No:
1137/003

EVALUATION CRITERIA

After 72 h grading, all erythema and oedema grades at 24 h, 48 h and 72 h were totalled for each test item extract and control for each individual animal. For calculating the score of a test item and control on each individual animal, the derived value was divided by 15 (3 scoring periods x 5 test or control sample injection sites). To determine the overall mean score for each test item and each corresponding control, the scores for the 3 animals were added and divided by three. The final test item score was obtained by subtracting the score of the control from the test item score.

Vehicle	Mean Reaction Score for test item extract	Mean Reaction Score for vehicle control	Overall difference (Test extract - control)
Physiological saline	A	B	(A-B)
Coarsest oil	C	D	(C-D)

If the difference between the mean reaction grades (erythema/oedema) for the test item and the control is greater than 1.0, then the test item was considered to cause intracutaneous reactivity.

RESULTS

Mortality and morbidity

No morbidity or mortality occurred in any of the animals throughout the experiment.

Body weight

An increase in body weight was observed in all the animals at the end of the experiment. Individual body weight of the animals is given in Table 1.

Clinical observation

No signs of ill health or overt toxicity occurred in any of the test animals.

Scoring of skin reaction

Injected sites appeared normal immediately after the injections. Grading of skin reactions for individual animals are given in Table 2 and Table 3. The difference of the mean skin reaction scores for the test item extracts and the vehicle control was zero (Table 4).

CONCLUSION

Based upon the results obtained and in line with ISO 10993-23:2021, the given test item, Hemorrhoid Ligator, supplied by Texplore Co., Ltd., is considered as non-reactive in New Zealand White rabbits under the conditions of the present study.

Irritation

FINAL REPORT
Rectal Irritation Test of Hemorrhoid Ligator in New Zealand White rabbits as per ISO 10993-23:2021

Study No:
1137/004

RESULTS

Mortality and morbidity

No morbidity or mortality were observed in any of the experimental animals.

Body weight

An increase in the body weight was observed in all the animals at the end of experiment (Table 1).

Clinical observation

No signs of ill health or overt toxicity, no signs of irritation around the rectal opening were observed.

Gross pathology examination

No gross lesions or no macroscopic evidence of irritation related to test item extracts was observed in the rectal tissue of the animals (Table 2).

Histopathology examination

Based on the histopathology grading (Table 3 & 4), the irritation index obtained was within the acceptable limit.

CONCLUSION

Based upon the results obtained in this study and performed in line with ISO 10993-23:2021, the given test item, Hemorrhoid Ligator, supplied by Texplore Co., Ltd., is considered as non-irritant to rectal mucous membrane in New Zealand White rabbits under the conditions of the present study.

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- ISO 13485 - Medical devices: Quality management systems

Certificate of Registration
 This certificate has been awarded to
M.E. MEDITEK CO., LTD.
 665 On-Nuch Road, Suan Luang District, Bangkok, 10250, Thailand
 74/9 Moo 22, Suwinthawong Road, Bang Nam Prio District,
 Chachoengsao, 24000, Thailand
 in recognition of the organization's Quality Management System which complies with
ISO 13485:2016
 The scope of activities covered by this certificate is defined below
Please refer to the Appendix

Date of Issue of Certification Cycle	Issue Number	Certificate Expiry Date	Certification Cycle
04 May 2021	3	03 May 2024	2
Revision Date	Revision Number	Original Certificate Issue Date	Scheme Number
04 May 2021	0	04 May 2018	n/a

For detailed explanation for the data fields above, refer to <http://www.urs-holdings.com/hogov-and-regulations>

Issued by  On behalf of the Schemes Manager

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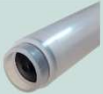
Clinical Study

- More than 200 cases are applied without any complication.
- Characteristic Feature & benefit

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